


Why would I choose the J Clamp over other clamps?

- **More comfortable**
- **Durable (others say to replace theirs every 90 days)**
- **Easy to use (one handle adjusts size and pressure)**
- **Easy to maintain (others have to drain & air after washing)**
- **Doesn't cause circulation problems because of the design to protect the blood vessels of the penis**
- **Smaller than other clamps. This allows urination without pulling down pants.**
- **Fits most penis sizes, between 2-4.5 inches circumference**
- **Requires very little dexterity**
- **This clam is 2 inches wide, 1.25 inches tall and 1 inch from the front to the back (some other clamps are wider than 3 inches)**

JMP
Jackson Medical Products, LLC

The J Clamp

External Male Incontinence Clamp

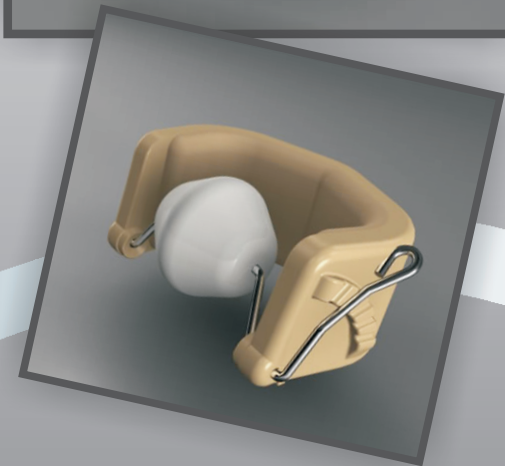

**The hygienic alternative to
diaper usage for adult males**

To order go to
www.JacksonMedicalProducts.com
or call 1-877-453-3531

What is an incontinence clamp?

The J Clamp is an external medical device used to control urinary leakage. This is accomplished by compressing the urethra to stop the flow of urine. This clamp is placed on the shaft of the penis then adjusted to the setting that is needed to stop urinary leakage.

How long can I wear the clamp each day?


Some manufacturers say 15-18 hours daily. The inventor of the J Clamp has worn the clamp 20 plus hours daily since February 2010. It is advised that each individual consult his physician.

Why would I choose the J Clamp over an adult diaper?

Hygiene is the main reason. Second is the cost of wearing diapers and the disposal of them especially in public places.

Will anyone know that I am wearing the J Clamp?

No. The J Clamp is small and easily passes through the zipper opening of pants. When in a public restroom, at the urinal, no one can detect that you are wearing the device. Other clamps, on the market, have to be taken off to urinate and some are so large that they will not pass through the zipper opening, so your pants have to be pulled down to urinate.

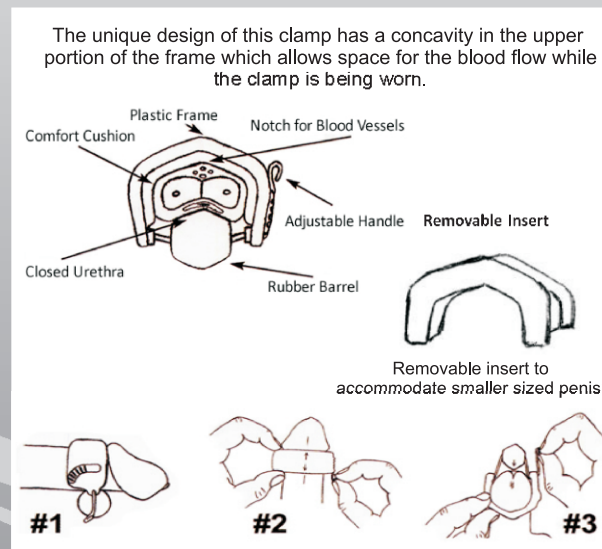

Care and Cleaning: DO NOT SCRUB PRODUCT
Wash the clamp with mild soap and water and blot dry.
30 Day Guarantee
The "J" clamp is guaranteed to be free from defects.
If there is a problem contact us immediately @
www.jacksonmedicalproducts.com or call 1-877-453-3531
or 615-453-3531
It is not guaranteed against normal wear or damage from abuse.
FDA Registered • Patent Pending

This product is made of durable nonporous bio compatible materials.

This clamp is easy to use even if you have dexterity problems. One handle controls size and pressure adjustments. The clamp is opened by rotating the adjustable handle counter-clockwise. With the adjustment handle on the right side of the penis, insert the penis. (Figure #1) through the clamp centered right to left. (Figure #2) Butt the front of the clamp against the head of the penis. Holding the clamp with the left hand and holding the adjustable handle with the right hand, rotate the handle clockwise until the rubber barrel contacts the underside of the penis. Turn the penis up to make sure the that the rubber barrel is centered right to left, against the underside of the penis. (Figure #3) Move the handle clockwise until the handle engages the first adjustment notch on the clamp. If you are able to urinate, continue to the next notch, continue adjusting clockwise until urine flow is stopped. If there is discomfort, move adjustment handle counter-clockwise to reduce the pressure. To urinate or remove the J Clamp, move adjustable handle counter clockwise until the clamp is opened.